

ADVANCED BED AND CHAIR MONITOR

A patient who moves around a lot may move off of the sensor pad for a few seconds, causing a false alarm. The Advanced Bed and Chair Monitor features a Delay control that helps prevent false alarms when the patient momentarily moves off the sensor pad, and an easy access reset button. These monitors also allow connection to your nurse call system, a call cord for increased notification capabilities, and AC power.



The Nurse Call jack lets you connect the monitor directly to your nurse call system.



The strain relief helps prevent damage to sensor pad cords.



Switches inside the battery compartment allow you to set the delay length and turn the pause feature on and off.

EASE OF USE

- ▶ Dedicated button for quick access to volume control
- ▶ Reset button allows your staff to quickly cancel an alarm (the monitor reactivates automatically without an On/Off switch)
- ▶ “Low Battery/Replace Pad”, “Alarm”, and “In Use” indicator LEDs show status at a glance

FLEXIBILITY

- ▶ Alarm delay setting helps prevent false alarms when the patient momentarily moves off the sensor pad
- ▶ Pause feature gives the caregiver 30 seconds to reposition the patient without activating the alarm
- ▶ Nurse call jack allows direct notification to nurse call
- ▶ Call cord jack allows the connection of call cords
- ▶ Volume can be turned off to reduce alarm fatigue
- ▶ AC power option extends battery life, reducing maintenance and replacement costs
- ▶ Bed, chair, and wall mount options are available (see reverse)

DURABILITY

- ▶ Strain relief helps prevent damage to the sensor pad cord
- ▶ Durable thick plastic casing and impact-resistant rubberized edging reduces replacement/repair costs
- ▶ Two year warranty

SPECIFICATIONS

- ▶ Size: 5.3" H x 3.5" W x 1.7" D
- ▶ Power Source: 4 AA batteries (included) with AC backup (optional)

COMPATIBLE SENSORS

- ▶ See reverse



SENSORS

BED AND CHAIR SENSOR PADS

To meet the needs of different patients and situations, Curbell's sensor pads are available in a variety of sizes and styles.

- ▶ Larger pad sizes provide more surface area for monitoring restless patients, thus reducing nuisance alarms
- ▶ Smaller pad sizes allow for more immediate alarm notification if the patient shifts out of normal position
- ▶ Three usable lifespans:
 - 14 and 30 day pads (white: suitable for hospital patients)
 - One year pads (blue: for long-term patients)
- ▶ Two styles: Corded, and Corded Timed



CORDED

Corded pads are a good choice when you need a simple sensor for your monitors. The breakaway cable allows connectors to separate under strain to minimize cord and monitor damage.



TIMED

An internal clock counts down the life of the pad based on actual use (i.e. when the pad is plugged in and pressure is applied to the pad), allowing you to extend the usable period of the pad. The monitor will notify you when to replace the pad.

| Lifespan | Pad Style | 7" x 15" Chair Pads | 10" x 15" Chair Pads | 4" x 30" Bed Pads | 10" x 30" Bed Pads | 20" x 30" Bed Pads |
|----------|---------------------|-----------------------|------------------------|-----------------------|--------------------|--------------------|
| 14 Day | Corded (white) | CSP-C14-NC7 (10 pack) | — | CSP-B14-NC4 (10 pack) | — | — |
| 30 Day | Corded (white) | — | CSP-C30-NC10 (10 pack) | CSP-B30-NC4 (10 pack) | — | — |
| One year | Corded (blue) | — | CSP-C365-NC10 | CSP-B365-NC4 | CSP-B365-NC10 | — |
| One year | Corded Timed (blue) | — | CSP-C365-TC10 | — | CSP-B365-TC10 | CSP-B365-TC20 |



CSP-TS-1 TOILET SEAT SENSOR

The Toilet Seat Sensor mounts discreetly underneath the toilet seat so that a patient can be monitored without a loss of privacy.



CS-SB2054V SEAT BELT SENSOR

The Seat Belt Sensor features a Velcro® fastener. In addition to setting off an alarm when unfastened, it also helps keep the patient from sliding down the chair.



CS-FMS-C2448 FLOOR MAT SENSOR

The 24" x 48" Floor Mat Sensor can alert staff if a patient attempts to stand or walk.



CS-FCS-C3668 FLOOR CUSHION SENSOR

The 36" x 68" Floor Cushion Sensor provides extra protection with 2" of padding in case the patient falls while attempting to stand or walk.

ACCESSORIES

| Model | Description | Model Number |
|---------------------------------------|---|--------------|
| Bed Mounting Bracket | Adjustable to fit 7/8" to 2 5/8" width headboards and footboards | CSM-BMB |
| Magnetic Mounting Bracket | Mounts to standard metal door jambs | CSM-MMB |
| Wall Mounting Bracket | Mounts to the wall with screws or Velcro strips | CSM-WMB |
| Velcro Strap | Replacement for any monitor | CSM-VS |
| AC Adaptor | The AC Adaptor keeps batteries charged up to three years | CSM-AC |
| Classic Call Cord | This 8' call cord plugs into the monitor to give the user the ability to call the nurse | CC96-010 |
| Jumper Cable | This 8' jumper cable connects the monitor to the nurse call system | JU96-295 |
| Replacement Bed Pad Breakaway Cable | 80" Breakaway Cable for 30 day and one year corded bed pads | CSP-BCR |
| Replacement Chair Pad Breakaway Cable | 34" Breakaway Cable for 30 day and one year corded bed pads | CSP-CCR |